 Référence Product number Riferimento	 Code du lot Batch code Codice lotto
 Date d'expiration Use by date data di scadenza	 -20°C Conserver à température ≤ -20°C Upper limit of temperature ≤ -20°C Limite massimo di temperatura ≤ -20°C
 Fabricant Manufacturer Produttore	 Contient des produits dérivés de sang ou de plasma humain Contains products derived from human blood or plasma Contiene prodotti derivati dal sangue o dal plasma umano

UPR/PUL/FAB/FI/025 - Ve 1 - 12/06/2026

SeroQal MALARIA

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1. INTENDED USE AND INDICATION

SeroQal MALARIA 101 and 103 are samples for laboratory use characterized as positive for Malaria antibodies.

2. COMPOSITION ET PACKAGING

Composition	Sample	Product number	Packaging
Human serum/plasma collected from blood donors	SeroQal Malaria 101	229592	1 cryotube, 3mL/tube - Label with barcode
	SeroQal Malaria 103	285763	1 cryotube, 1,5mL/tube - Label with barcode

The human sera and plasma composing SeroQal are controlled with respect to markers HBs Ag, HBc Ab, HCV Ab, HIV p24 Ag, HV-1 and HIV-2 Ab, HTLV-I and HTLV-II Ab and Syphilis Ab.

Samples showing positive serology are treated with methylene blue to inactivate pathogens and thus reduce the risk of infection.

The prepared samples are filtered to a minimum of 0.45 µm (clarifying filtration) and are free of preservatives and antibiotics.

3. DELIVERY AND STORAGE CONDITIONS

Samples must be delivered at a temperature ≤ -20 ° C (controlled temperature or dry ice transport).

Before opening, samples must be placed and stored upon receipt in an enclosure ≤ -20°C. They are then stable until the expiration date indicated on the label.

After opening, the samples can be stored between +2°C and +8°C for 7 days or can be refrozen after use without exceeding 3 freeze-thaw cycles. Monitoring the number of cycles is the responsibility of the user.

4. PRECAUTIONS AND LIMITS FOR USE

SeroQal MALARIA 101 and 103 use is strictly reserved for professionals trained and authorized to perform analyses and to handle of biological samples.

Bad transport, storage or use conditions can lead erroneous results. It is recommended not to use and dispose of the samples if conditions are not guaranteed.

Do not use if tubes are damaged or if samples show physical alteration indicating possible instability or degradation of the product.

5. SAFETY INSTRUCTIONS

Products do not contain a hazardous ingredient, no substances with European Community workplace exposure limit values, or substances of very high concern (SVHC) in quantities greater than their respective reporting limit values. Consequently, the product is not supplied with a safety data sheet because the concentrations of chemicals contained in this product do not exceed the thresholds defined in the European Regulation 1907/2006.

Products are composed of blood products of human origin from donors subjected to mandatory blood screening tests according to French regulation. The wearing of personal protective equipment is recommended when handling the product.

Products are not subject to the regulations of Transport of Dangerous Goods with regard to the ADR and IATA except for UN1845 labelling for shipment in dry ice.

Products as well as all media that have been in contact with the control must be handled as potentially infectious products. Disposal conditions and disinfection procedures must comply with current regulations.

6. PROTOCOL

The samples are ready for use. They must be left at room temperature until completely thawed, then gently homogenized, avoiding foaming. The user recommendations for handling biological samples for analysis apply to SeroQal MALARIA 101 and 103.

7. PERFORMANCES

No specific usage technique is claimed. The samples were evaluated in production, and the following results were observed:

Sample	Référence	Malaria Ab DIAPRO	ELISA anti-plasmodium IgG EUROIMMUN
SeroQal Malaria 101	229592	$1,1 \leq R \leq 5,0$	NEGATIVE
SeroQal Malaria 103	285763	$R > 8,0$	$1,1 \leq R \leq 5,0$

8. DETECTION OF ANOMALIES

Any incident related to the device's performance and/or safety must be reported to the manufacturer as soon as possible.

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